Coding the meter

Code meter before using it for the first time and every time you change to another vial of OneTouch® FastTake® Test Strips.

STEP 1

Enter code mode.

Start with meter turned off. Insert OneTouch® FastTake® Test Strip to turn on meter. All display segments will appear followed by date and time. Code number will appear next.

STEP 2

Match code numbers.

Compare code number on meter display with code number on test strip vial. If they match, begin testing. If they do not match, follow Step 3.

STEP 3

Code meter.

Press C button until code numbers match. After  appears, OneTouch® FastTake® System is ready for testing.

BEFORE TESTING: See inside for instructions on coding the meter and the test procedure.

Important: Read detailed instructions in your OneTouch® FastTake® Owner's Booklet before doing a test.

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ONE TOUCH
FastTake

Display symbols and messages

Blood glucose test result with suggestion to check ketone levels.

Average of last 14 days (64 tests performed).


Blood glucose level higher than 33.3 mmol/L (600 mg/dL).

Blood glucose level lower than 1.1 mmol/L (20 mg/dL).

Temperature too low for testing.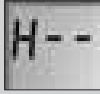

Temperature too high for testing.

© LifeScan, Inc. 2003
Millitas, CA 95035
10/2003

Exit memory mode.
Press M button to turn off meter.

STEP 3

After three seconds, most recent test result with date and time appears. Press C button once and next most recent test result will appear. Continue pressing C button to recall last 150 test results in order.

STEP 2

Recall test results.
Touch blood drop to TOP WHITE EDGE of test strip. Do not push your finger against test strip. Hold blood drop to TOP WHITE EDGE of test strip until formation window.

STEP 1


Insert test strip.
Meter turns on automatically.

STEP 3

Apply sample.
Touch blood drop to TOP WHITE EDGE of test strip. Do not push your finger against test strip. Hold blood drop to TOP WHITE EDGE of test strip until formation window.

STEP 2

Accurate results in just 15 seconds.

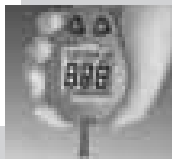test summary

Using the meter memory

Battery power too low for testing. Replace batteries immediately.
Note: Millimoles per litre (mmol/L) is the standard unit of measure in Canada. Select mmol/L whenever you reset the meter.


Problem with meter.*

Meter electronics problem.*

Blood sample applied before  appeared on display.*

Test strip damaged, removed during test, or inserted improperly.*

* Retest with a new test strip. If problem persists, call the OneTouch® Customer Care Line at 1 800 663-5521.

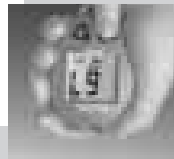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