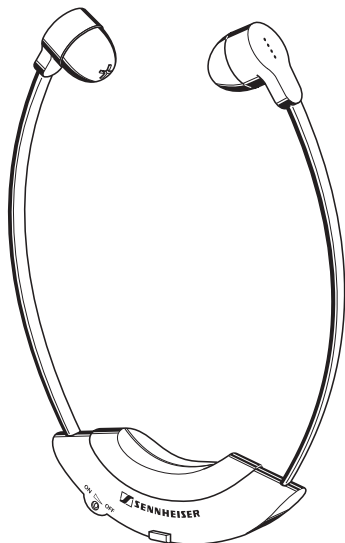
GEBRAUCHSANLEITUNG
INSTRUCTIONS FOR USE
NOTICE D'EMPLOI
ISTRUZIONI PER L'USO
INSTRUCCIONES PARA EL USO
GEBRUIKSAANWIJZING


RI 150
RI 300

Gebrauchsanleitung	3
Instructions for use	11
Notice d'emploi	19
Istruzioni per l'uso	27
Instrucciones para el uso	35
Gebruiksaanwijzing	43

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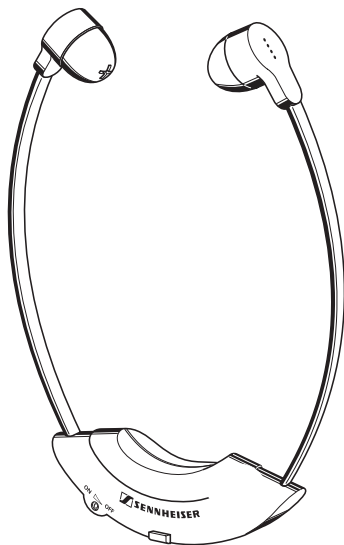RI 150
RI 300

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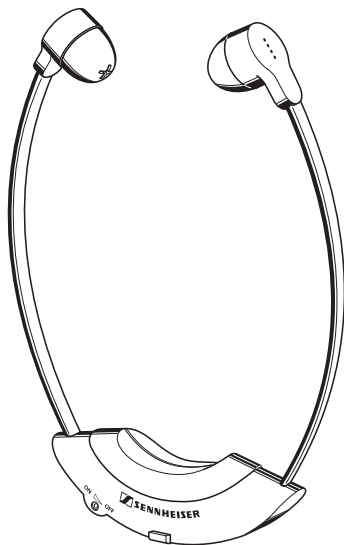
RI 150
RI 300

THANK YOU FOR CHOOSING SENNHEISER!

We have designed this product to give you reliable operation over many years.

Please take a few moments to read these instructions carefully, as we want you to enjoy your new Sennheiser product quickly and to the full.

AREAS OF APPLICATION

Infra-red stethoset receivers for use as additional receivers for existing infra-red transmission systems or for use in conferences and simultaneous translation applications.

VARIANTS

► RI 150

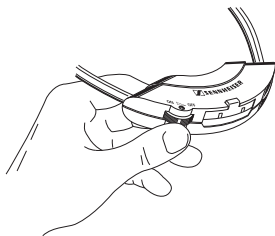
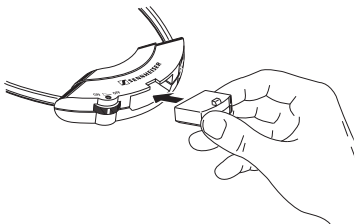
Infra-red mono stethoset receiver, designed for use as a personal sound amplifier for people with impaired hearing. It can also be used as an additional receiver for the Set 150 infra-red mono transmission system.

► RI 300

Infra-red stereo stethoset receiver, designed for use as a personal sound amplifier. It can also be used as an additional receiver for the IS 300 infra-red stereo transmission system.

ADVANTAGES

- With cordless sound transmission, you can be independent from cables!
- According to current scientific knowledge, sound transmission by infra-red light poses no risk to health whatsoever.
- High reception reliability due to the carrier frequencies of 2.3 MHz and 2.8 MHz. Fluorescent tubes and digital controls do not use this frequency range.
- Lightweight and extremely comfortable to wear - the receiver weighs only 45 g, including the rechargeable accupack!


PUTTING THE RECEIVER INTO OPERATION

Insert the BA 151 accupack into the compartment in the receiver.

SWITCHING ON THE RECEIVER ON

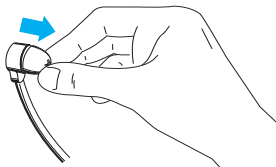Use the rotary switch to switch on the receiver and adjust the volume.

CLEANING / REPLACING THE EAR-CUSHIONS

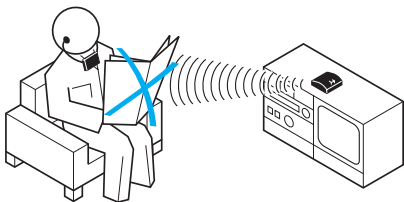
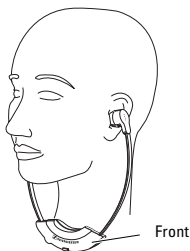
Use a damp cloth for cleaning your stethoset receiver. Do not use any solvents.

The ear-cushions can easily be removed from the headphones and cleaned with warm soapy water. From time to time, however, the ear-cushions should be replaced for reasons of hygiene. Replacement ear-cushions are available with different degrees of softness:

- ▶ Black ear-cushions, firmer version EP1 Cat. no. 04655
- ▶ White ear-cushions, softer version EP2 Cat. no. 04656

Please order your ear-cushions from your Sennheiser dealer.

Download from [Www.Somanuals.com](http://www.somanuals.com). All Manuals Search And Download.

FREE LINE OF SIGHT

Infra-red light is transmitted in the same way as the light from a light bulb. Do not obstruct the path of the light between the transmitter and the receiver, and stay in the room where the transmission is taking place!

Make sure that you wear the stethoset receiver the right way round, with the “Sennheiser” logo at the front.

There is a simple rule: If you can see the transmitter or the infra-red radiator from where you are, transmission can take place without any interference.

VOLUME UP? - NO!

When people use headphones, they tend to choose a higher volume than with loudspeakers. Listening at high volume levels for long periods can lead to permanent hearing defects. Please protect your hearing, Sennheiser headphones have an excellent sound quality even at low volumes.

ACCUPACK OPERATION

- ▶ When the accupack becomes weak (noise interference and sound distortion are sure signs of a weak accupack), switch off the receiver and recharge the accupack. (If necessary, use a second accupack to listen while the first accupack is being recharged.)
- ▶ Recharge the accupack immediately when it is flat. If the accupack is flat but is left in the switched on receiver, this reduces the service life of the accupack (“total discharge”).
- ▶ As far as possible, use the full capacity of an accupack before you recharge it. Frequent short discharge and recharge cycles prematurely reduce its charging capacity.

CHARGING THE ACCUPACKS

The stethoset receivers are powered by BA 151 accupacks which can be recharged either separately or in the receivers. Depending on the application, there are different possibilities for recharging the accupacks.

The BA 151 can be recharged

- ▶ in the Set 150 and Set 300 transmitters,
- ▶ in the L 151-2 charger (suitable for charging two individual accupacks) or
- ▶ in the L 151-10 charging unit.

For commercial use, Sennheiser offers charging and transport/storage units for recharging a larger number of stethoset receivers at a time.

The BA 151 accupack must be charged for 24 hours before first use! Please refer to the corresponding section in the operating manual of your infra-red transmission system or the operating manuals for the chargers.

ENVIRONMENT AND HEALTH

Devices powered by rechargeable accupacks are kind to the environment. This energy source can be recharged many times. Conventional batteries, on the other hand, must be disposed of as special waste after they have been used just once. If an accupack should be defective, it is then recycled - the dealer who sold it to you will take it back and will also supply you with a new, original Sennheiser accupack. (Recycling methods differ in different countries).

RECHARGEABLE ACCUPLUGS HAVE A „CHARGING CYCLE“

Depending on how long your receiver has been in use, you must also recharge the BA 151 accuplug for a certain time. A simple rule of thumb will help you to decide what time is necessary:

Operating time x 3 = recharging time

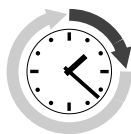
Example:

1 hour listening
3 hours recharging

2 hours listening
6 hours recharging


3 hours listening
9 hours recharging


OPERATION WITH A SECOND ACCUPLUG

By using a second accuplug, you can increase the listening time with your receiver. The second accuplug is always inserted in the charging compartment of the transmitter to ensure that it is ready and fully charged when it is needed.

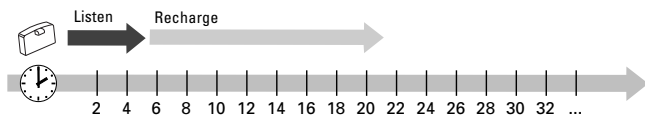- ▶ If you use your system with only one accuplug, you must stop after about five hours and recharge the accuplug for about 15 hours in the charging compartment of the transmitter.

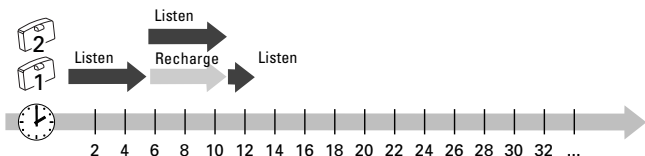- ▶ With a second accuplug, you can listen for more than twice the time. Simply place the discharged first accuplug back into the charging compartment. After the (partial) recharge, there is an additional (partial) amount of power available:

IMPORTANT!

- ▶ Keep the receiver away from central heating radiators and electric heaters. Never expose it to direct sunlight.
- ▶ Use the receiver in dry rooms only.

TECHNICAL DATA

	RI 150	RI 300
Modulation	FM, mono	FM, stereo
Carrier frequencies	2.3 MHz	2.3 MHz / 2.8 MHz
AF frequency response	18-18,000 Hz	18-18,000 Hz
THD	< 1 %	< 1 %
Signal-to-noise ratio	typ. 70 dB	typ. 64 dB
Operating time with one accupack	approx. 6 h	approx. 5 h
Weight with / without accupack	45 g / 35 g	45 g / 35 g
Max. sound pressure level	130 dBA	130 dBA / 126 dBA

SENNHEISER TECHNOLOGY AND SENNHEISER SERVICE ARE FUTURE-ORIENTED


Even after many years, you will still have unlimited pleasure from your infra-red sound transmission system. You can buy replacements for wearing parts such as ear-cushions and accupacks for many years to come. And in the unlikely event that any repairs should be necessary, Sennheiser service departments worldwide give moderately priced, fast and competent repairs.

ACCESSORIES

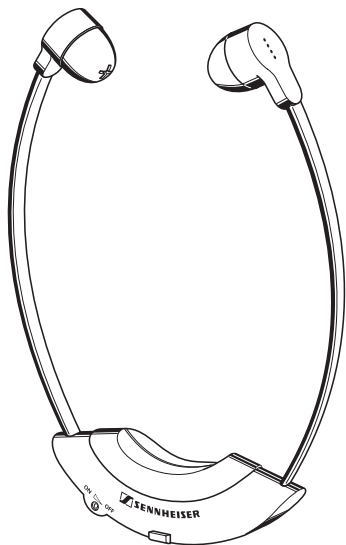- ▶ BA 151 accupack Cat. no. 04146
- ▶ Black ear-cushions, firmer version EP1 Cat. no. 04655
- ▶ White ear-cushions, softer version EP2 Cat. no. 04656
- ▶ L 151 charger for two accupacks Cat. no. 04457
- ▶ L 151-10 charging unit for 10 receivers or 10 accupacks Cat. no. 04494

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Sennheiser Irland GmbH